


STEP 1

First wash your hands and sit down in a chair with a low backrest. Please make sure you have read the user instructions completely before using the Eyot®.




STEP 2

Unscrew the cap of the bottle / tube. Push the dropper completely into the Eyot®.

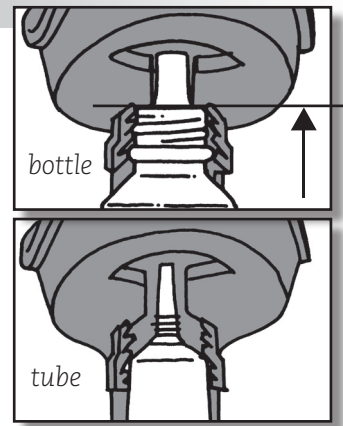


STEP 3

Place the lower support of the Eyot® just below your lower eye lid. Gently push the eyelid downwards until the Eyot® rests on your cheekbone.

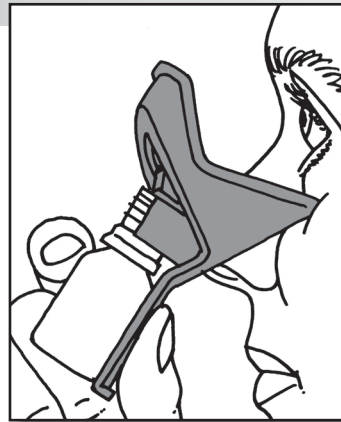



STEP 4

Then place the upper support of the Eyot® on your forehead. The device and the bottle are now in the correct position to your head.

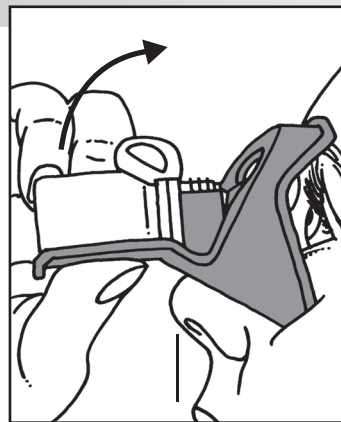

STEP 5

Now look up through the window in the upper support of the Eyot®. Tilt your head backwards until you can see the ceiling straight above you. You may find this easier if you close your other eye.

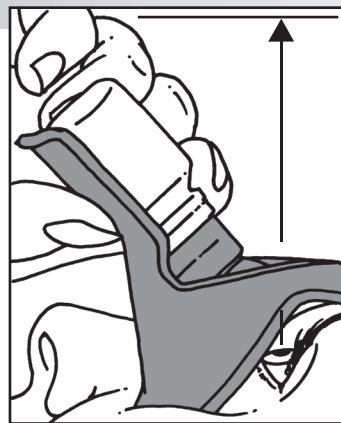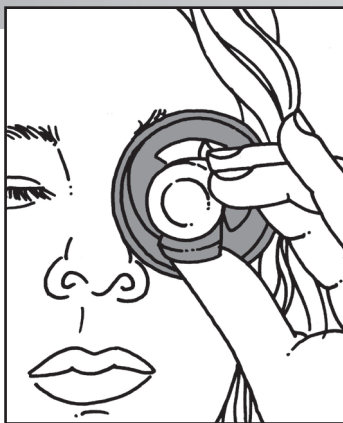

Keep this position and squeeze into the bottle / tube until one drop has fallen into your eye. Repeat this process for your other eye if required. When done, remove the bottle out of the Eyot® and close it.

Carefully read these instructions before using the Eyot®. It is important that you use your eye drops regularly as recommended by your doctor. Further information about the use of your eye drops can be found on the consumer medicine information leaflet enclosed with each bottle.

User instruction


Eyot®

EYE DROPPER DEVICE

MANUFACTURER AND DISTRIBUTOR: ITEMNUMBER 6300
SPRUYT HILLEN
TINBERGENLAAN 1
3401 MT IJSSELSTEIN
THE NETHERLANDS



EYOT® IS A REGISTERED TRADE NAME.
PATENTS PENDING.

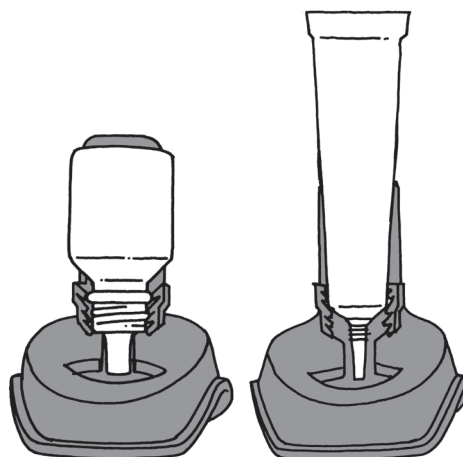

PRODUCT INFORMATION

Eyot® is an universal eye dropper device which is only compatible with soft plastic eye dropper bottles / tubes. Eye ointment tubes are not suitable for the Eyot®.

Eye dropper bottles / tubes made of glass or hard plastic are not suitable for the Eyot®.
Do not use eye dropper bottles with a dropper shorter than 8 mm and longer than 14 mm.

Eyot® is made from a recyclable thermoplastic synthetic material
Eyot® measures up to the EC guidelines 93/42 for Medical devices Class 1

CLEANING ADVISE

After using the Eyot®, rinse in lukewarm water. Dry It with a clean cloth. Please retain the Eyot® dust free.
Do not use hot or boiling water and do not clean the Eyot® in a dishwasher.
Please use the Eyot® for private use only and not for other purposes.

