



Eyot[®]

AID FOR ADMINISTRATION OF PRESERVATIVE FREE EYE DROPS IN SINGLE-DOSE CONTAINERS SUPPLIED BY PHARMACEUTICALS LIMITED. THIS AID IS DESIGNED TO HELP YOU POSITION THE UNIT DOSE CONTAINER CORRECTLY FOR SUCCESSFUL APPLICATION OF YOUR EYE DROPS.

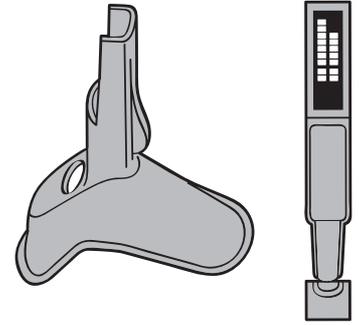

Carefully read these instructions before using the Eyot. It is important that you use your medicine as recommended by your doctor. Further information about the correct use and storage of your Eye Drops can be found on the Patient Information Leaflet enclosed with each medicine. This device provides a simple, alternative, instillation technique.

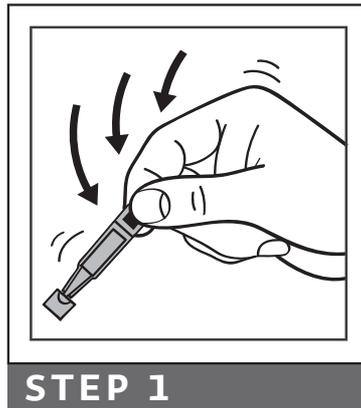


STEP 1

Wash your hands and sit in a chair with a low backrest. Make sure that the fluid is near the opening of the single-dose container. You may need to shake the container gently.



STEP 2

Twist off the top of the single-dose container.

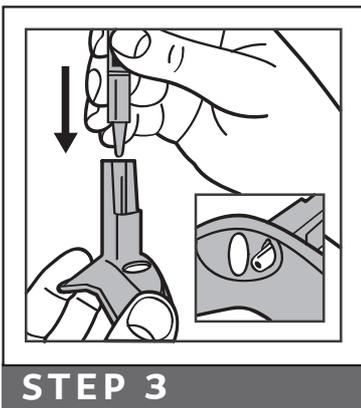


STEP 3

Push the single-dose container, nozzle end first, firmly downwards into the device until it is in position. Make sure the dispenser nozzle tip is pushed fully through the device (see smaller picture above). Check this before each use.




STEP 4

Hold the Eyot as shown in the above picture (the small opening should be above the single-dose container tip). Place the lower part of the device just below your lower eyelid and pull the eyelid gently downwards with the device until there is a "pocket" between the eyelid and your eye.

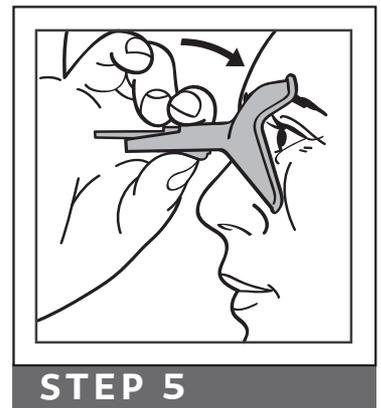


STEP 5

Place the upper part of the device near your eyebrow, against your forehead.

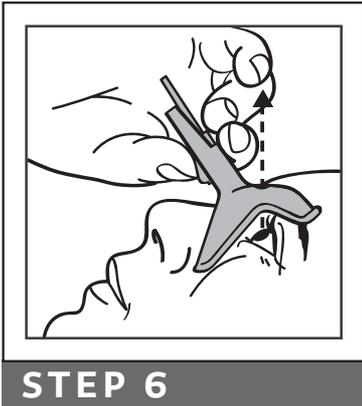

STEP 6

Look through the small opening in the device. Tilt your head backwards until you can see the ceiling straight above you through the opening. Your head is now in the correct position to administer your eye drops.

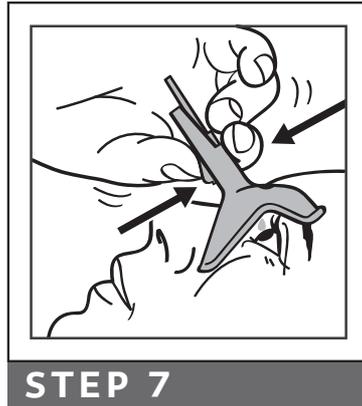

STEP 7

Squeeze the device carefully with your fingers. The eye drop should fall directly into your eye. Take the device away from your eye and press a finger against the corner of this eye next to your nose. Hold for 1 minute whilst keeping the eye closed. You should repeat the process for your other eye, if directed by your doctor.



STEP 8

Remove the single-dose container from the device and throw it away. Do not keep it to use again even if there is liquid left in it because sterility cannot be maintained after opening. Open a new single-dose container each time.

CLEANING ADVICE

Remove the single-dose container from the device and wipe the device with a clean dry cloth. If necessary you can rinse the device in lukewarm water and then dry it with a clean cloth. Do not use hot or boiling water. Do not clean the device in a dishwasher.

OTHER INFORMATION

The UD Eyot should only be used in combination with preservative-free products packed in single-dose containers. This device provides a simple, alternative, instillation technique. The UD Eyot is made of a recyclable thermoplastic synthetic material. This Device conforms to European Demands for Medical Devices, Class I.

WARNING

Check before each use that the single-dose container is properly positioned in the device. Moreover, make sure the single-dose container tip is pushed fully through the device.

ADDRESS

Manufacturer and distributor: Spruyt hillen BV, Tinbergenlaan 1, Postbus 46, 3400 AA IJsselstein, Phone +31 30 2814411.

